



Siesta External Auxiliary Battery Pack

Extend the operational time of the **Siesta 802** and **Safiro** digital amplifier recording systems with the **External Auxiliary Battery Pack**. The battery pack is an available accessory that plugs into the power port of the **Siesta 802** and **Safiro** recorder units to provide up to 90 hours of operation on one charge. Sparing the use of the internal battery pack within the recorder, the Auxiliary Battery Pack can be “hot-swapped” with another pack to provide multi-day uninterrupted recordings.

An external charger is used to recharge the **Auxiliary Battery Pack** in less than 4 hours. The Auxiliary Battery Pack is contained in a soft pouch designed to attach to the waist belt most commonly used with the Siesta/Safiro Pouch. The pack has a display port exposing multi-colored LEDs that indicate the power level and recharging status of the Battery Pack so the user knows when to recharge the pack and when it is ready for use.




S I E S T A[™]
802

How it works

To use the Siesta with the **External Auxiliary Battery Pack** connect the pack to the **Siesta** using the easy-to-use plugs.

The plugs just push together and pull-apart where the arrows 'line-up'

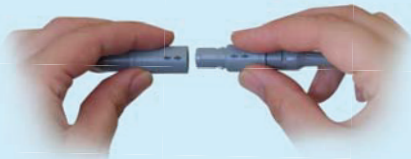

When the Battery Pack is charged it will display a **green light** in the clear window of the casing when connected to the Siesta.



When the Battery Pack requires charging it will display an **orange light** in the clear window of the casing.

To re-charge the **External Auxiliary Battery Pack**, just plug it into the charger using the same easy-to-use plugs. The charger and **Battery Pack** will indicate with a **green light** when it is ready for use.

The re-chargeable batteries are enclosed in an internal case within the Battery Pack so that the batteries can be charged easily during a patient study.



SPECIFICATIONS

- The package weighs about 400 grams (14.11 ounces) with dimensions 225 x 100 x 30mm (8.85 x 3.93 x 1.18 inches)
- Rechargeable NiMH battery cells with a smart circuit providing power and charging status
- Smart Charger - (50/60Hz, 110/220VAC), weighs about 250 grams (8.81 ounces) with dimensions (115mm x 60mm x 55mm) 4.25 x 2.36 x 2.16 inches.

PART NUMBER

Part Number	Description
8007-0034-01	Kit-Siesta External Auxiliary Battery Pack with Charger
8007-0035-01	Siesta External Auxiliary Battery Pack in Pouch
0407-0007-00	Charger for External Auxiliary Battery Pack

Contact your Compumedics representative for pricing and ordering information.

All specifications are subject to change without notice. Please contact your Compumedics representative for latest technical information, pricing and product availability. Siesta, COMPUMEDICS and the Compumedics logo are registered trademarks of Compumedics Limited.

For more information please contact:

Compumedics Limited, Australia:
Headquarters
 30-40 Flockhart Street
 Abbotsford VIC 3067, Australia
 Ph: +61 3 8420 7300
 Fax: +61 3 8420 7399
 Free Call: 1800 651 751

Compumedics USA, Inc.:
 5015 West WT Harris Blvd, Suite E
 Charlotte, NC 28269
 Toll Free: +1 877 717 3975
 Ph: +1 704 749 3200
 Fax: +1 704 749 3299

Compumedics Europe GmbH:
 Werdauer Strasse 1 - 3
 01069 Dresden
 Ph: +49 351 5019-7682
 Fax: +49 351 5019-7684

Compumedics France SAS:
 Rue Jean Sapidus,
 Bât Pythagore
 67400 Illkirch-Graffenstaden
 Ph: +33(0) 981 062 869
 Fax: +33(0) 970 604 963



www.compumedics.com