

onSIGHT™ A.V.S

AMBULATORY VIDEO EEG SOLUTION



Monitor your patient's home ambulatory studies with CONFIDENCE!

EEG with Video recording in "Real Time"!

- > True web application for remote monitoring
- > Access through any commercial browser
- > View all data on computer or tablet
- > Office, home, or mobile access

By applying the very latest in technological innovations, Compumedics is now able to provide truly informative real-time interaction with ambulatory recordings. OnSight recordings are collected and reviewed using Compumedics Profusion EEG software, the same software used in our sophisticated laboratory systems around the world.



High Definition Camera

- > Integrated infrared light source
- > 150 degree image framing
- > 30 frames per second

Easy to transport

- > Reinforced wheeled case with retractable handle

Time Savings

- > Simple set-up: 5 minutes to enter patient data and program internal memory card
- > Less than 20 minutes download time (USB3) for typical 72 hour EEG/video recording (60GB)

Simple Two-Step Recording Start

1. Patient plugs the ONsight system into a standard home power outlet
2. Extend the camera mount and point the camera towards patient

Recording Duration Options

Full 24 hours - 4 internal lithium AA batteries
Up to 72 hours - with external battery pack.
 Your patient never has to interrupt the recording to change the batteries.



Protective camera cover included



Data Security

- > Two copies of recording are preserved - Internal memory card and PC
- > HIPAA 256 AES encryption
- > Viewed over secured cellular network



Patient is wearing Okti recorder

Case Dimensions	Cm	Inches
Case	35 x 57 x 23.5	13.77 x 22.44 x 9.25
With Handle Extended	35 x 86 x 23.5	13.77 x 33.85 x 9.25
Camera Pole Extention	113 High	44.48 High
Weight	Kg	lbs
	< 11.52	< 25.4

Compumedics Profusion EEG is the true next generation clinical and LTM software package that offers unprecedented ease of use both in the EEG lab, and in post-recording analysis and review. Developed in close consultation with EEG clinicians and neurologists, it is designed for easy navigation and editing of up to 45 days of continuously recorded EEG.

Fully web-based management tool that provides seamless, automatic data archiving, study status tracking, complete program metrics and anytime, anywhere data access with remote live review via all major browser interfaces. Profusion EEG review software is integrated into the Nexus 360 application.
The perfect way to optimize your new ONsight program!